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Use:
Air Walking Boot provides stabilization and protection of foot and ankle joint area. Follow-up treatment in case of stabile fractures of ankle/instep.

Contraindications:
The aircells must not be overinflated as it may lead to skin irritation. Special attention should be paid in case of diabetic patients and patients with sensory disturbances. In case of skin changes or irritations, the compression should be decreased.
The product cannot be used in case of open, unstable fractures, difficult to heal wounds, sores and other skin changes. It is necessary to contact a doctor in case of pain or numbness.
Special attention should be paid while walking on wet or slippery surfaces. It is necessary to contact a physiotherapist or a doctor before changing the way of using the brace.
The product should be used only under careful medical supervision.

- Placement instruction:**
- Untie strap and soft foam liner, position foot carefully into soft foam liner, step heel back. Steady position ❶.
 - Make sure the ankle is well positioned inside the walker, fasten the hook and loop fasteners of the soft foam liner ❷.
 - You may put pad kits to supplement inner space or between walker frame and soft foam liner if needed ❸.
 - Place front panel in correspondent place ❹.
 - Fasten all the straps from a ankle proximal to distal and adjust the straps to comfortable tight ❺.
 - The front toe protector can be adjusted to fit required foot length. to adjust the toe protector, put it in upright position❻. Unlock by pressing both ends and adjust it to the required length ❼ and adjust it to the suitable fixed holes. After adjusting put the toe protector in the flat position ❸.
 - To inflate aircell, turn the button ❹ towards the indicated “inflate” direction until it is thight, then press the black pump button ❿ to inflate air util aircel feels snug. Stop inflating while uncomfortable compression felt. Heavy compression may reduce blood circulation.
 - To deflate aircell, turn the button ❹ towards indicated “deflate” direction until it is tight, the aircell will deflate automatically. To accelerate aircell deflating, press the black pump button ❿.

Washing instructions:
Remove foam liner for hand wash, using a mild detergent and cold water, air dry. Do not wash in washing machine or in mechanical dryer. For plastic components, clean with a damp cloth, air dry. Do not use detergent or bleach.

Maintenance:
The inner liner should be washed using sponge and mild detergent. (but is should not soak through). Dry far away from artificial source of heat. Dry in a flat position. Do not iron. The cover should be cleaned with a moist cloth.

- CAUTION**
- If discomfort or pain results, persists or increases, discontinue to use and consult with medical professional.
 - Make sure the hook and loop fasteners are well fastened while wearing the product. If the hook and loop fasteners can not be Fastened well, stop using the product to prevent injuries.

THE FINAL DECISION CONCERNING THE USAGE AND SELECTION OF THE APPLIANCE SHALL BE MADE BY THE DOCTOR SPECIALIZING IN THE FIELD

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