Keep away from artificial sources of heat. Check the condition of the brace after each wash. Clean the product systematically, at least twice a month, and more often if necessary.

Materials:

PA, PES, PU, PE, AL.

ATTENTION!

Do not use the device if you have circulatory disorders.
Do not use the device if it has been damaged or deformed.
Do not attempt to repair the device yourself.
Do not use the device in the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor.

ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

ATTENTION!










Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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The figures below show how to apply the brace to the right hip:

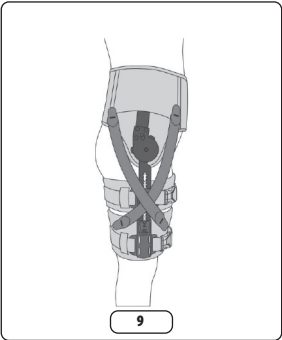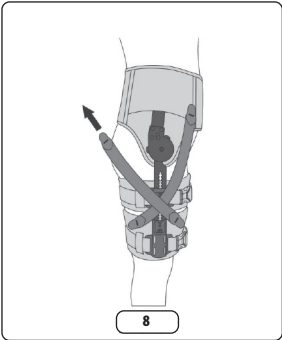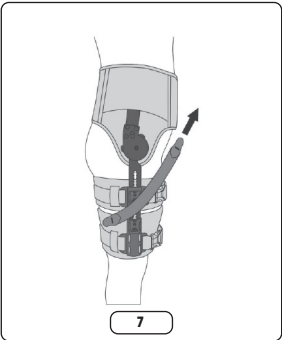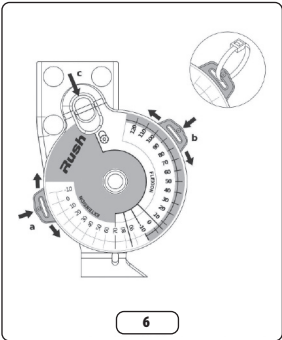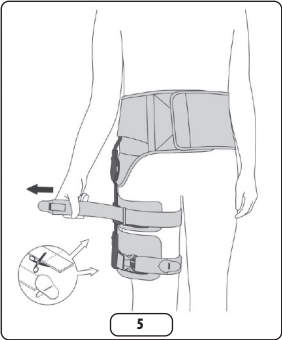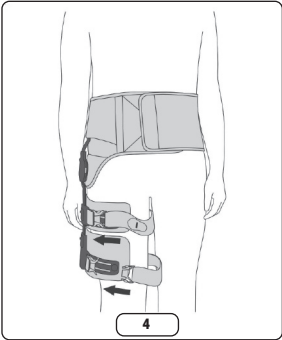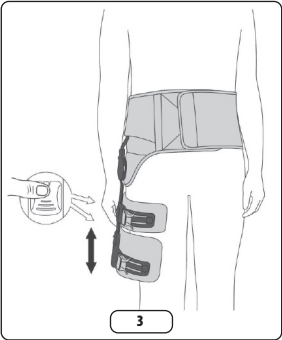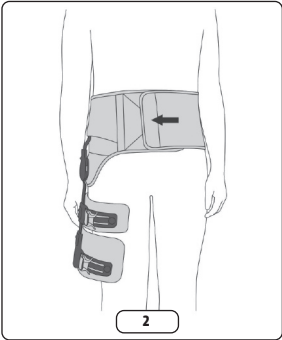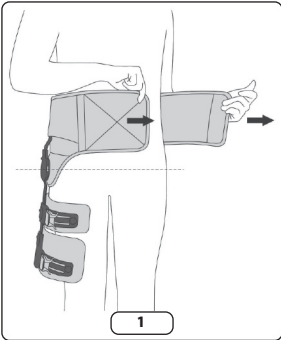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- Limits adduction and abduction and controls adduction and abduction movement of the leg at the hip joint.
- Supports and stabilises the joint.

Indications:

- Hip osteoporosis.
- Hip dislocation.
- After surgeries, such as arthroscopy and joint replacement.

Contraindications:

- Hip fractures.
- Conditions requiring total hip immobilisation.
- Hard-to-heal wounds or skin lesions within the area of the brace.
- Allergic reaction to the materials used.

Intended user:

The medical device is intended for laypersons (non-professional users).

Target groups of patients:

The brace is designed for patients with hip joint dysfunction, following injury or disease, to support the healing and rehabilitation process by limiting adduction and abduction and controlling adduction and abduction movement of the hip joint.

Disposal of the medical device:

The device must be disposed of following the applicable local environmental regulations.

Choosing your brace:

- The first time the brace is fitted, it should be done upon prescription and under the supervision of qualified medical personnel. The doctor or physiotherapist should decide how much stabilisation and compression the hip joint needs
- The brace should be selected based on the dimensions shown in the size chart.
- The package includes two elastic waist belts to choose from (a longer and shorter one).
- If the product becomes too loose or changes its position, it should be reattached starting from the first step in the „How to put on the brace” section.

How to put on the brace:

The brace is initially locked in the 0° position; the length of the brace is between 49 and 56 cm. Optimal fitting of the brace by qualified medical personnel is essential to ensure that your hip is properly stabilised.

The brace includes additional straps to increase the strength of leg abduction. FIG 7-9

- Before putting on the brace, undo the snap buckles.
- Attach the waist belt to the Velcro at the back. Fit the brace to the involved leg, ensuring that the centre of the angle adjustment chuck is at the level of the hip joint. FIG. 1
- Tighten and fasten the belt around the hips at the front. FIG. 2
- The bottom liner has two independent grommets. Pressing the button at the centre of the grommet, grab the grommet and extend it to the desired length. Repeat the same for the second grommet. FIG. 3
- Fasten the buckles. FIG. 4
- Tighten the straps using the required compression and fasten the Velcro straps. The length of the belt can be adjusted as shown in the drawing. FIG. 5
- Make sure the brace fits properly. Note: If it turns out that the brace needs to be extended or shortened, unfasten the buckle at the selected strap and move the grommets to the required position, tightening the straps if necessary.
- To set the desired range of joint mobility, release the Quick Lock (c) FIG.6. The lock allows the brace to be immobilised at angles of 0°, 15°, 30°, 45°.
- Press the flexion (b) and extension (a) angle lock to set the angle of your choice. Use cable ties to prevent accidental pressing of the lock. FIG. 6
- If necessary, attach additional straps with Velcro to increase the abduction strength of the leg. FIG. 7-9

Size chart:

Universal size for right and left leg.

Size	Universal
Hip circumference (cm)	75 - 135

Instructions for cleaning:

Detach the liner elements from the stabilising splint before washing. Lining elements and tightening straps should be hand washed at low temperatures (30°) with mild detergents, without softeners. Do not use bleach or strong detergents. Do not spin, do not wring, gently squeeze before drying. To dry, spread the parts of the upholstery on a towel and air-dry.