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EN

Indications:

The DUO brace is intended to stabilise the fingers of the hand. It is intended for use in deformities and injuries of the finger joints, chronic joint instability, traumatic injuries to tendons, recovery of injuries after removal of plaster cast, recovery of injuries after surgery.

Contraindications:

- Skin lesions, edema, varicose veins, blood clotting disorders.
- Allergic reaction to materials used.

Intended user:

The medical device is intended for the general public (a non-professional user).

Target patient groups:

It is intended for patients with deformed and damaged finger joints, chronic joint instability, traumatic injuries to tendons, for the recovery of injuries after removal of plaster cast, for the recovery of injuries after surgery.

Disposal of the medical device:

Disposal of the device must be carried out in line with the applicable local environmental regulations.

How to put on the brace:

The brace provides immobilisation of two adjacent fingers of the hand in various configurations (not applicable to the thumb). The liner can be placed either on the top of the hand or on the bottom.

Brace selection:

1. Before first use, check that the size of the brace is correct. An incorrectly fitted brace can result in reduced therapy, cause discomfort and lead to incorrect stabilisation of the fingers.
2. The device should be selected by a doctor or physiotherapist.
3. The brace should be selected based on the dimensions shown in the size chart.
4. If the device becomes too loose or changes its position, it must be refitted starting from the first step in the section, How to put on the brace’.

Size chart:

Universal size, suitable for right and left hand.

Size	Wrist circumference
Universal	15-26 cm

Cleaning procedures:

The metal rails must be removed from the brace before washing. The brace must be washed by hand at low temperatures (30° C) with mild detergents, without fabric softeners. Do not use bleach or strong detergents. To dry it, lay the brace spread out on a towel and air-dry. Do not twist or wring the brace. Do not dry in a tumble dryer or on a heater. Check the condition of the brace after each wash. Cleaning of the device should be carried out systematically, at least twice a month, and more often if necessary.

Composition:

PES, PA, PU, AL

CAUTION!

In case of a skin allergy, stop using the device immediately and consult a doctor. Do not use for more than 24 hours continuously. Do not use the device on children without adult supervision. Do not use the device if it has been damaged or distorted. Do not attempt to repair the device yourself. Do not use the device on patients with superficial and deep sensation disorders without guardian

supervision. Do not use the device in the wrong size or in a way for which it is not intended.

ATTENTION!

The product is designed to be worn directly on the hand. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the brace with warming gels or ointments. Any serious incident that occurs in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

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