#### CONTRINDICATIONS:

- Unstable fractures and vertebrae dislocation which require surgery,
- serious injuries resulting from inflammations and cancer diseases,
- vertebra deformations and herniation of the nucleus pulposus which require immobilization,
- post-surgery conditions, especially with the use of bone graft.

#### CORSET ADJUSTMENT AND REGULATION

##### Warning:

- Safe and effective corset adjustment can be performed only by qualified personnel, under doctor's control,
- the most appropriate position to put on the corset is lying,
- height and width regulation should be performed once the corset is taken off the body,
- elements of the corset are bound together with bolts (loosing or changing the setting of the bolts allows to regulate the height of lateral part and the width of thoracic part of the corset),
- all the frame regulations (width and height) should be performed symmetrically,
- it's possible to adjust the corset to the patient's body by bending the thoracic arches and rear pad.

#### ADJUSTMENT AND FITTING:

1. Choose the size (please, check the size chart).
2. Put on the corset – the front pad should touch the upper edge of sternum manubrium (junction of clavicle and sternum).
3. Make sure that the bottom pad rests on the on the hypogastrium (underbelly), above the pubic symphysis;
4. Make sure that the lateral pads (upholstered elements) adhere to the trunk and the hinges should be at the height of anterior superior iliac spine, in midaxillary line, below iliac crest.
5. Adjust the rear pad to the pathological symptoms according to doctor's advice.

Size	S	M	L
Distance between sternum manubrium to pubic symphysis (cm)	37 - 44	41 - 48	45 - 52

#### CONSERVATION AND MAINTENANCE:

Product hygiene is important for patient's rehabilitation. Corset elements made of eco-leather can be cleaned with gentle soap and water. The corset should be dried before using. It can be cleaned with gentle detergents. Please, do not use the strong chemical detergents and bleaches.

#### ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric. The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING THE APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A CONSULTANT

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